See English version on page 9

BIOVET

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Contenu de la trousse

- Formulaire Biovet

The image shows a 'Request form' for an 'All-in-One Microbial Test'. The form is divided into several sections: 'SUBMITTER INFORMATION' (with fields for name, address, phone, fax, and email), 'ANIMAL IDENTIFICATION' (with fields for species, breed, sex, age, and sample collection type), 'HEALTH HISTORY' (with checkboxes for various conditions), and 'LABORATORY INFORMATION' (with checkboxes for specific tests). The form also includes a QR code and logos for 'MIDOG' and 'Biovet'.

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Storage: Room temperature

NOTICE

The MiDOG® Comprehensive Microbiome Test is not intended to be health information or medical data or to be used to screen, diagnose, treat, prevent, or assess risk of any disease or condition. The MiDOG® service is a bacterial and fungal identification test grounded on scientific research that uses specific sample collection, nucleic acid extraction, Next-Generation Sequencing, and bioinformatic technologies to ascertain the composition and identity of bacteria and fungi as well as antibiotic resistance in collected samples. The MiDOG® service has not been cleared or approved by the U.S. Food and Drug Administration.

Read all instructions before use.

CAUTION: Urine Conditioning Buffer™ preservation reagent is provided in the urine collection tube. DO NOT drink, touch, or remove the preservation reagent from the swab collection tube. The preservation reagent is an irritant to the skin and eyes. Do not ingest preservation reagent; avoid contact with skin and eyes; wash with water for 20 minutes if exposed to skin or eyes. Do NOT induce vomiting if the preservation reagent is ingested. Obtain appropriate medical attention in the event of any exposure. Keep out of reach of children.

Urine Kit Components

- Request Form

The image shows a request form for an 'All-in-One Microbial Test'. The form is titled 'Request form' and includes sections for 'SUBMITTER INFORMATION', 'ANIMAL IDENTIFICATION', 'HEALTH STATUS', and 'TREATMENT HISTORY'. It contains various checkboxes and text boxes for recording patient and sample details. At the bottom, there are logos for 'MIDOG' and 'Biovet'.

- Urine Collection Tube

Also includes a shipping bag (ziploc) and an instruction leaflet.

Urine Collection Kit - Instruction

2. Label swab collection tube with the names of the veterinarian and the dog. Please make sure that you are using the collection tube with the yellow label for your sample collection.

Sampling Examples

Urine collection via cystocentesis

2a. Collect the urine from your patient using cystocentesis.

Urine collection via free-catch

3a. Use a sterile container to catch the urine from the patient.

2b. Add the collected urine from the syringe into the MiDOG® urine collection tube.

3b. Pour the collected urine into the provided MiDOG® urine collection tube.

Use one collection tube for each patient.
Please provide as much urine as possible using the MiDOG® urine collection tube. Ideally provide at least 2 ml of urine.

Also available: **Swab collection kit**

Prepare sample for shipping

8. Screw the cap securely onto the swab collection tube. Ensure the tube is sealed tightly to prevent leakage.

9. Shake the swab collection tube 10 times to fully cover the swab with solution.

Shipping samples

10. Place collection tube into shipping bag (Ziploc) and DO NOT put the request form into the same pouch.

11. Sent your samples to Biovet by mail or courier. If your in the province of Quebec, you can ask for a pickup.

Samples can stay at room temperature, so no icepack is needed.

After the sample is received, your MiDOG® Report will be sent to the email address you provide within 8 business days.

To contact us

BIOVET[®]

1-888-824-6838

sales@biovet-inc.com

4375 Beaudry, Saint-Hyacinthe QC J2S 8W2

Rev. MKT-057-V.1 - 2024-01-22